

CONTENTS OF PACKAGE.

- PYtest® capsule.
- Balloon.
- Paper cup (x2).
- Straw.
- Package seals.
- Return carton.

PREPARATION PRIOR TO TEST.

- Ensure that the patient has NOT test:
- Antibiotics for 4 weeks prior to the test.
 - Sucralfate for 2 weeks prior to the test.
 - Losec, Zoton, or Somac for 1 week prior to the test.
 - Food or drink (including water) for 4 hours (recommended 6 hours) prior to the test.


Should the patient be unsure of the above preparation, consultation with a doctor is advised. Continue if preparation is complete.

- Ensure the patient is seated during the test and have a time piece available.
- Label the balloon with patient name, date and start time.
- If applicable, make up the reply paid box (postage already paid). This will allow the patient to insert the PYtest® balloon after the test has been completed.
- Should you have concerns regarding your current medication and the impact of this on the test, please refer to table which lists "medications which restrict the usefulness of the test" (shown on the right)

DIRECTIONS TO THE PATIENT

1. Sit down, tip the capsule out and swallow it whole. Wash down the capsule with water from the cup supplied (fill to 2/3 full).
2. Three (3) minutes later fill (to 2/3 full) the paper cup again with water and drink it.
3. Wait for seven (7) minutes sitting down.
4. Take the straw and insert it in the slit, found on the neck of the balloon. Take a deep breath, hold for approximately 5-10 seconds then blow slowly through the straw into the balloon until it is firm. If the balloon is not filled with the first attempt, simply continue to blow until firm.
5. Hold the neck of the balloon at the base firmly. remove the straw, twist the neck of the balloon a few times and tie a knot in the neck of the balloon (carefully ensure this knot is tight).
6. Record the finish time on the balloon and return to testing laboratory immediately after test is completed.

Distributed by :
All Eights (M) Sdn Bhd (98420 T)
45 Jalan TS 6/10A, Subang Industrial Park, 47610 Subang Jaya, Selangor Darul Ehsan.
Tel: (603) 5633 4988 Fax: (603) 5633 0261
Email: all8@alleights.com.my

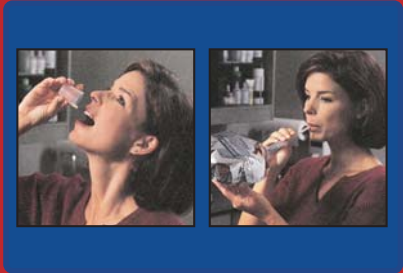

THE FOLLOWING MEDICATIONS SHOULD NOT BE TAKEN PRIOR TO THE TEST.

MEDICATIONS WHICH RESTRICT THE USEFULNESS OF PYtest

Exclusion period	Type of medication	Generic	Trade name
During Fasting and During test	Antacids	Al-U-Tab	Gabul
		Andren-Turns Antacid	Medobam 2
		Arkosid	Mucoside
		De Wits Antacid Powder	Mucoside
	H2 Antagonists	Dacal	Mucoside
		Erio	Mucoside
		Geonigel	Stigma Liquid Antacid
		Gavison	Tralac
	Cimetidine	Cimetidine	Signilidine
		Cimetidine	Tagamet
		Cimetidine	Tagamet
		Cimetidine	Tagamet
	Ranitidine	Ranitidine	Raninac
		Ranitidine	Raninac
		Ranitidine	Raninac
		Ranitidine	Raninac
	Famotidine	Famotidine	Famotidine
		Famotidine	Famotidine
		Famotidine	Famotidine
		Famotidine	Famotidine
	Nizatidine	Nizatidine	Nizatidine
		Nizatidine	Nizatidine
		Nizatidine	Nizatidine
		Nizatidine	Nizatidine
For 1 Days Prior to Test	Proton Pump Inhibitors	Zoton	Zoton
	Omeprazole	Omeprazole	Omeprazole
		Omeprazole	Omeprazole
		Omeprazole	Omeprazole
		Omeprazole	Omeprazole
For 14 Days Prior to Test	Cytoreductives	Succinate	Succinate
	Antibiotics	Amoxycillin	Amoxycillin
		Amoxycillin	Amoxycillin
		Amoxycillin	Amoxycillin
		Amoxycillin	Amoxycillin
	Tetracycline	Tetracycline	Tetracycline
		Tetracycline	Tetracycline
		Tetracycline	Tetracycline
		Tetracycline	Tetracycline
	Bismuth	Bismuth	Bismuth
		Bismuth	Bismuth
		Bismuth	Bismuth
		Bismuth	Bismuth

Note: All antibiotics except Vancomycin and Sulfa (Sephin, Bacrim etc) should be withheld for 4 weeks for maximal accuracy

PYtest®

For the diagnosis of

Helicobacter pylori

PYtest® (Carbon-14-Urea 1.0 microCurie Breath Test) **MAL08032202AZ**

For the Detection of *Helicobacter pylori*
Manufactured by : Avent, Inc
66205, Memorial Place, Suit 100, Tucson, AZ85756, USA

Contents - 1 PYtest® Capsule & PYtest® Breath Collection Accessories

CAUTION RADIOACTIVE MATERIAL
[Physical Half Life 5730 yrs, 156keV (max) beta emission]

Controlled Medicine / Ubat Terkawal

Keep Out of Reach of Children
Jauhkan Daripada Kanak-kanak

For *in vivo* oral administration.
For dosage information, please see Product Information
Caution: Federal Law Prohibits Dispensing without Prescription
Store below 30°C
U.S. Patent No. 4830010

Lot:
Mfg:
Exp: