


PPD Information Box		ALRP Information Box	
<b>Technical Information:</b> <span style="display: inline-block; width: 10px; height: 10px; background-color: black; margin-right: 5px;"></span> Black <span style="display: inline-block; width: 10px; height: 10px; background-color: gray; margin-right: 5px;"></span> Cool Gray 11 <span style="display: inline-block; width: 10px; height: 10px; background-color: red; margin-right: 5px;"></span> Warm Red <span style="display: inline-block; width: 10px; height: 10px; background-color: green; margin-right: 5px;"></span> 368 <span style="display: inline-block; width: 10px; height: 10px; background-color: orange; margin-right: 5px;"></span> 485 <span style="display: inline-block; width: 10px; height: 10px; background-color: purple; margin-right: 5px;"></span> 2597	<span style="display: inline-block; width: 10px; height: 10px; border: 1px solid black; margin-right: 5px;"></span> DIE CUT <span style="display: inline-block; width: 10px; height: 10px; border: 1px solid black; margin-right: 5px;"></span> Coating Free <span style="display: inline-block; width: 10px; height: 10px; border: 1px solid black; margin-right: 5px;"></span> PERFORATION <span style="display: inline-block; width: 10px; height: 10px; border: 1px solid black; margin-right: 5px;"></span> KISS CUT	<b>Translations of Variable Data</b> Lot: Lot Exp Date: Exp. Mfg Date: Mfg. date Price: N/A GTIN: N/A Serial Number: N/A	
<b>Previous Item Code (to be destroyed)</b> N/A	<b>Serialization Information</b> Type: N/A Code: N/A		

### Quick Reference Guide


These are not complete instructions. Read the full INSTRUCTIONS FOR USE.

**Step 1** Choose your injection site

You or another person can inject the medicine in your stomach (abdomen) or thigh.

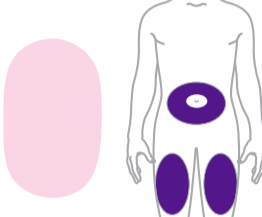


Another person should give you the injection in the back of your upper arm.



**Step 2** Pull off the gray base cap

Make sure the Pen is **locked**.

**Clear Base**




**Pull** the gray base cap straight off and throw it away in your household trash.

**Do not** put the gray base cap back on.

**Do not** touch the needle.


**Step 3** Place clear base on skin, then unlock

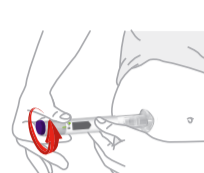

**Place** the clear base flat against your skin at the injection site.

**Unlock** by turning the lock ring.

**Step 4** Press and hold up to 10 seconds

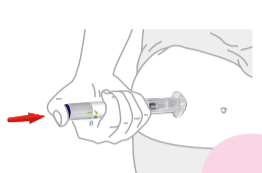


**Press and hold** the purple injection button for up to 10 seconds.

**Listen** for:


- First click = injection started
- Second click = injection completed

Injection is complete when you see the gray plunger.



**After your injection**

**Place** the used Pen in a sharps container. See **Disposing of your used Pen** in the full INSTRUCTIONS FOR USE.

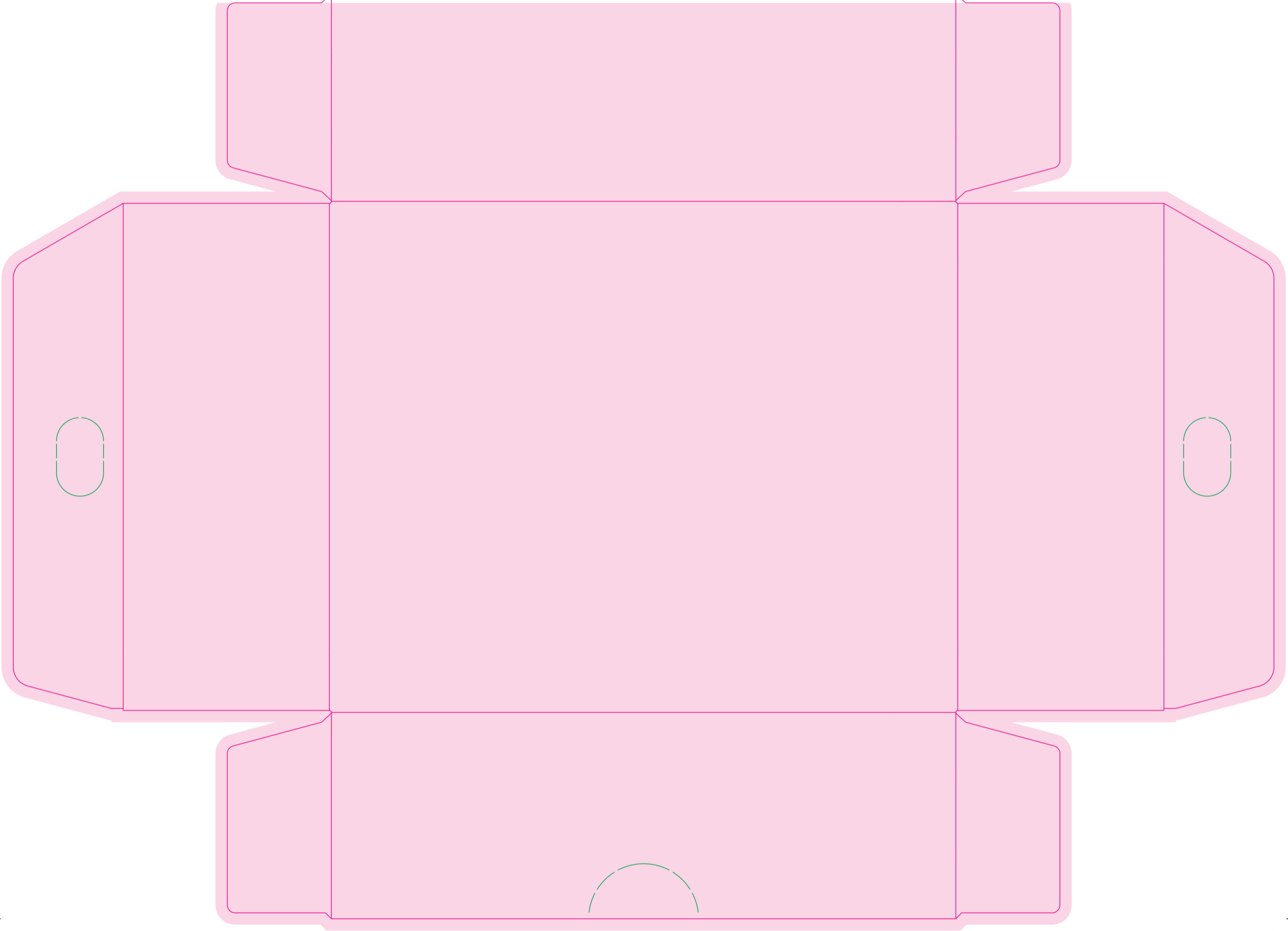

PPD Information Box		ALRP Information Box	
<p><b>Technical Information:</b></p> <ul style="list-style-type: none"> <li><span style="display: inline-block; width: 10px; height: 10px; background-color: black; margin-right: 5px;"></span> Black</li> <li><span style="display: inline-block; width: 10px; height: 10px; background-color: #90EE90; margin-right: 5px;"></span> 368</li> <li><span style="display: inline-block; width: 10px; height: 10px; background-color: #FF0000; margin-right: 5px;"></span> 485</li> <li><span style="display: inline-block; width: 10px; height: 10px; background-color: #4169E1; margin-right: 5px;"></span> 2597</li> </ul>	<ul style="list-style-type: none"> <li><span style="border: 1px solid black; display: inline-block; width: 10px; height: 10px; margin-right: 5px;"></span> DIE CUT</li> <li><span style="background-color: #FFDAB9; display: inline-block; width: 10px; height: 10px; margin-right: 5px;"></span> Coating Free</li> <li><span style="border: 1px dashed black; display: inline-block; width: 10px; height: 10px; margin-right: 5px;"></span> PERFORATION</li> </ul>	<p><b>Translations of Variable Data</b></p> <p>Lot: N/A</p> <p>Exp Date: N/A</p> <p>Mfg Date: N/A</p> <p>Price: N/A</p> <p>GTIN: N/A</p> <p>Serial Number: N/A</p>	<p><b>Serialization Information</b></p> <p>Type: N/A</p> <p>Code: N/A</p>
<p><b>Previous Item Code (to be destroyed)</b></p> <p style="text-align: center;">N/A</p>			
